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GB

1. Rights and responsibilities

It is the responsibility of the customer to carry out the Electrical (including the appropriate earthing), Multimedia, Pneumatic, Water supply and Waste installations, to the connection point of the KaVo product, using the services of a duly qualified person, according to the installation plans supplied by KaVo.

The maintenance of the installation cited in the previous item is the responsibility of the owner and/or user of the product.

It is not the responsibility of the authorised KaVo technician to measure or check the quality of the earthing (nor the rest of the installation).

If the technician finds a fault in the earth wiring, the product will not be installed.

The purchaser must pay for the installation charge and the travelling and overstay expenses of the technical staff involved with the installation service call outside the urban perimeter of the city where the authorised service is located.

As soon as these instructions are received, the customer should contact the authorised service for clarification.

2. Information on the installation for the purchaser

2.1 Electrical Installation

The distribution box has a connector where the power supply wires are connected. The purchaser must provide the electrical installation in accordance with this instructions.

The electrical installation prior to the installation of the product must be done in accordance with the standards in force at the site on "Low voltage electrical installations".

The wires of the power supply must have their capacity calculated taking into consideration the consumption of the product, the length of the wires and the maximum voltage drop.

Consumption: 280 VA
All KaVo products must be earthed;

The quality of the earthing implies the safety of the dentist, his or her auxiliary and the patient. The earthing (as well as the rest of the electrical installation which supplies the product) must be inspected, measured and corrected periodically, as it is subject to degeneration over time.

The earth wire must be identified with the green yellow colouring.
The neutral wire must be identified with the light blue colouring.
It is recommended that the supply cables are protected by a trip switch.

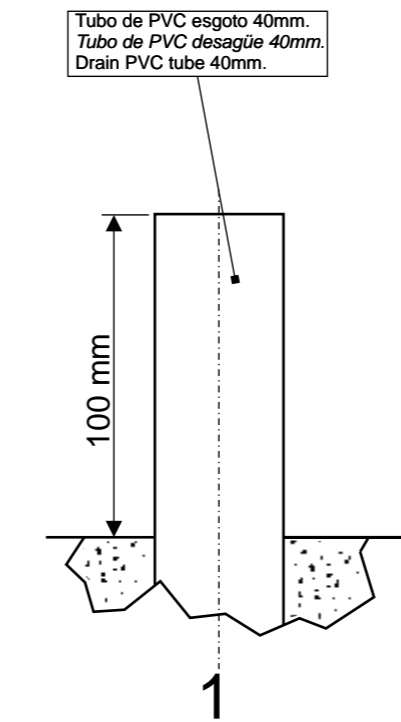
Multimedia kit

The cable signal with the video VGA (Computer) must have female connector, 60cm out of electrical conduit.
The other end (Computer), must have the output of the video signal in perfect condition.
The electrical conduit transmission should not have knees 90°, but curves 90°.

The KaVo assumes no liability for damage from poor quality of the installation.

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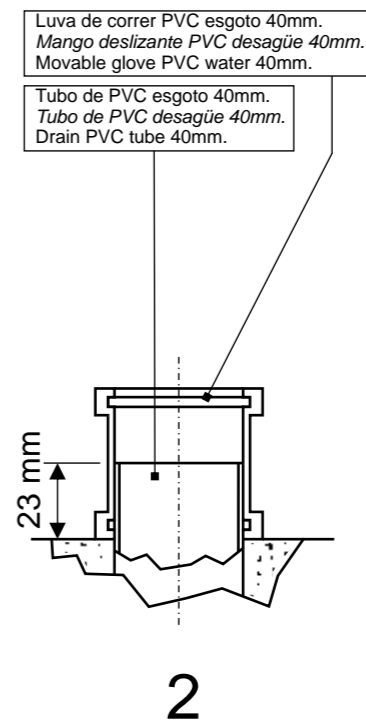
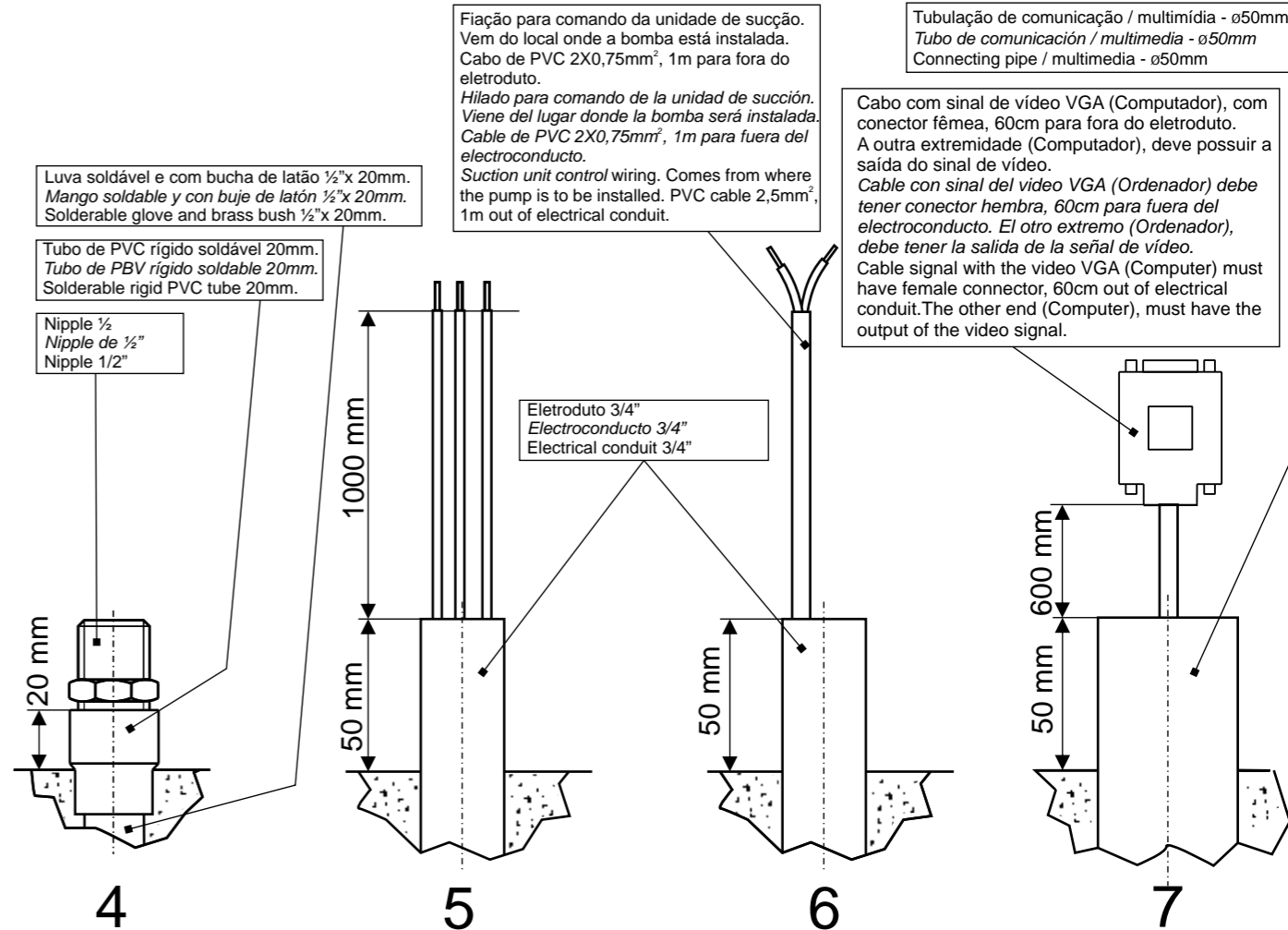
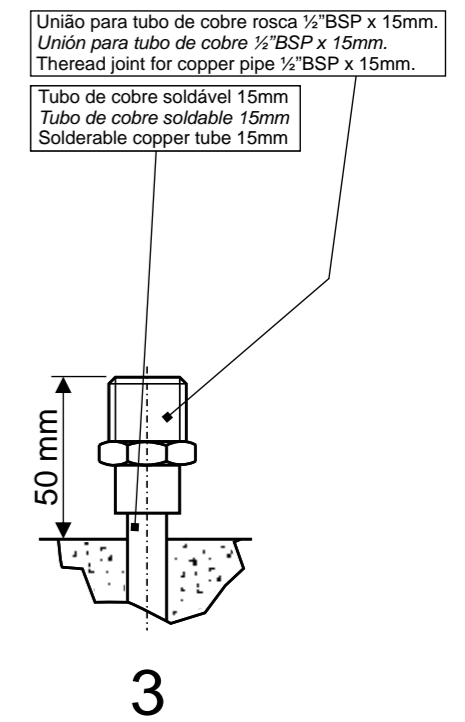
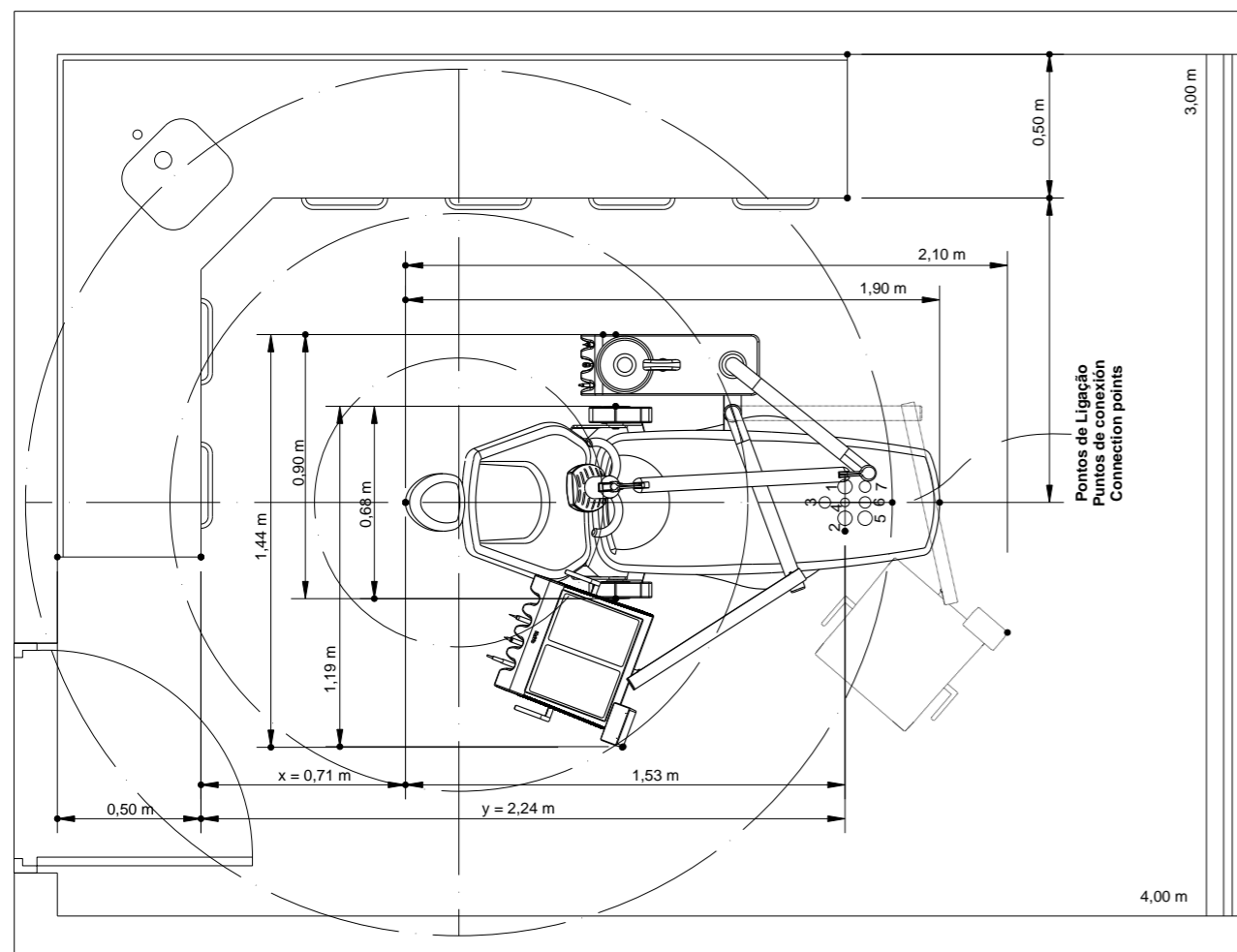
GB

**2.7 Area necessary for installation
Positioning of the connection point
between equipment and the supply**

The measures X and Y are guides and can be altered by the dentist so as to meet the needs.

Guidance values for a patient of 1.76 m in height, with the seat of the chair at a height approximate 0,58 m at its rear part and the backrest in the lying position. Measures in meters.

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GB

2.2 Pneumatic installation (air)

The air supplied to the equipment must be dry, and free of impurities and oil. It must be supplied at a pressure between 5,0 and 5,5 bar. The outflow must be at least 100 liters per minute at 5.5 bar. The air supply branch must have an air filter with a mesh not greater than 25µm, as per the standard ISO 7494-2. The shape and positioning of the tubing must be in accordance with the instructions contained on pages 4, 5, 6 and 7. In order to execute maintenance if there are two or more items of equipment linked to the supply branch, the buyer will be able to adapt an air register in the tubing which is waiting for maintenance. Before installing equipment, see the local legislation concerning air supply, mainly as regards its quality.

2.3 Water installation

Compliance with the values below indicated is an indispensable requirement for the perfect functioning of the KaVo equipment. The installation and positioning of the tubing must be in accordance with the instructions given in pages 4, 5, 6 and 7. For the purposes of maintenance if there are two or more sets of equipment connected to the same supply branch, the purchaser may adapt one water tap in the wait tubing. **As a safety measure the supply branch must have a general tap which can interrupt the supply when the equipment is not in use.** Before installing the equipment contact the water supply company and find out about the local dispositions concerning the water supply.

Quality of the water: drinkable water
Hardness of the water: 1.50 – 2.14 mmol/l
– 8.4 – 12 dH
Value –pH: 7.2 – 7.8
Water filter: 80 m
Supply pressure: 2.5 – 6.0 bar
Supply volume: 3.5 l/min

2.4 Waste

The form and positioning of the tubing must be in accordance with the instructions given in pages 4, 5, 6 and 7. The minimum drop is 10mm/m. Observe the standards in force for the locality on "Outflow of used water".

2.5 Suction

The form and positioning of the tubing must be in accordance with the instructions given in pages 4, 5, 6 and 7.

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2.6 Pattern for positioning the tubes at the connection point between the equipment and the supply.
Tubes under the floor.

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